Multiflex Ankle 30 mm

Instructions for Use

379342-379344

EN	Instructions for Use	3
FR	Instructions D'Utilisation	11
DE	Gebrauchsanweisung	19
IT	Istruzioni per L'Uso	27
ES	Instrucciones de Uso	35
NO	Bruksanvisning	43
RU	Инструкция протезиста	51
TR	Kullanım Talimatları	59

Blatchford:

1 Description and Purpose

Multiflex Ankle provides the user with comfortable multi-axial motion, which enhances stability. Changeable ankle ball and snubbers provide the ability to customize the ankle according to the user's weight and Activity Level.

Application

These instructions are for use by the practitioner.

This device is an ankle unit that is for use as a component of a lower limb prosthesis.

Activity Level

This device is recommended for Activity Level 1–3 users.

As an exception to our recommendation,

Activity Level 4* users may only use this device based upon the outcome of a prosthetic assessment.

*Maximum user weight: 100 kg (220 lb).

Contra-indications

This device may not be suitable for use in competitive sports events, as this type of user may be better served by a specially designed prosthesis optimized for their needs.

Intended for a single user.

Ensure that the user has understood all instructions for use, drawing particular attention to [section 4](#) regarding maintenance.


Ankle Ball and Snubber Selection

Snubbers are sold separately. For fitting advice see [section 7](#).

W = White **O** = Orange **B** = Blue

User Weight	60 kg (131 lb)	80 kg (176 lb)	100 kg (220 lb)	125 kg (275 lb)
Ankle Ball				
Ankle Part Number	379342	379342	379343	379344
Snubber				
Snubber Part Number	409007	409008	409008	409009

2 Construction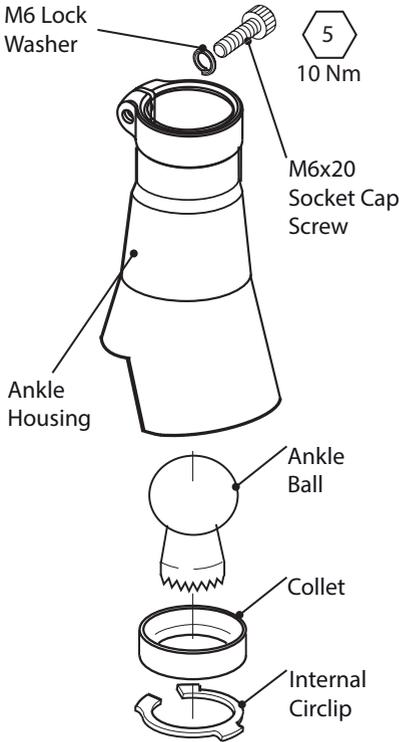

Principal Parts:

- Ankle Housing (aluminum)
- M6x20 Socket Cap Screw (plated carbon steel)
- M6 Lock Washer (plated carbon steel)
- Ankle Ball (aluminum/natural rubber)
- Collet (aluminum)
- Internal Circlip (plated carbon steel)

3 Function

Multiflex Ankle comprises an ankle housing with a rubber ball joint. When used with an appropriate snubber kit, these components provide the user with a resilient multi-axial ankle motion for secure foot placement at heel strike and a smooth progression through the gait cycle. The ankle ball primarily controls plantarflexion and the snubber controls dorsiflexion support.

4 Maintenance

Only competent personnel should carry out maintenance.

Carry out the following maintenance annually:

- Check the tightness of the socket cap screw
- Check the ankle for excessive play

Advise the user to report the following to the practitioner:

- Changes in body weight or activity level
- Changes in the performance of this device, for example:
 - Unusual noises or play
 - Stiffness
 - Increased range in motion

Cleaning

Use a damp cloth and mild soap to clean outside surfaces. **Do not** use aggressive cleansers.

5 Limitations on Use

Intended Life

A local risk assessment should be carried out based upon activity and usage.

Lifting Loads

User weight and activity is governed by the stated limits. Load carrying by the user should be based on a local risk assessment.

Environment

Avoid exposing Multiflex Ankle to corrosive elements such as water, acids and other liquids.

Avoid abrasive environments such as those containing sand, as these may promote premature wear.


Suitable for outdoor use

Exclusively for use between -15 °C and 50 °C (5°F and 122 °F).

We recommend using Blatchford products with this device.

6 Bench Alignment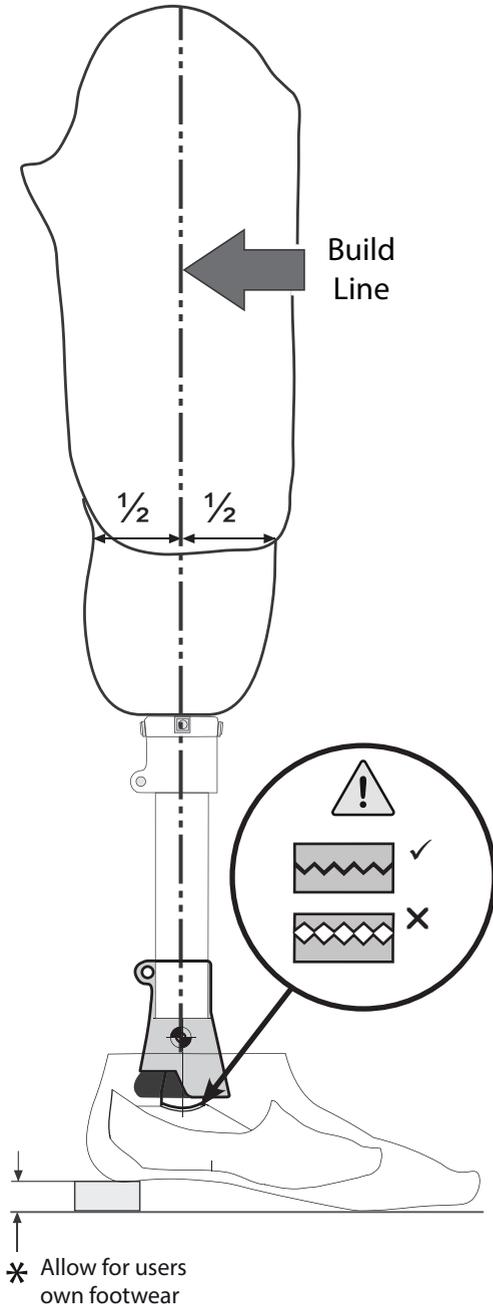

6.1 Additional Components

Follow the alignment instructions that are supplied with any additional components.

6.2 Static Alignment

With the flexion accommodated the build line should pass through the center of the socket in the sagittal (A-P) plane and through the center of Multiflex Ankle. Adjust the heel height at the serrated interface.

Make sure that, when the user is standing, the heel and the toe are evenly loaded and the foot is in full contact with the floor.

6.3 Dynamic Alignment

Coronal Plane

Minimise M-L thrust by adjusting the relative positions of the socket and the foot.

Sagittal Plane

Make sure that the user transitions smoothly from heel strike to toe-off.

- ⚠ Check that the serrations are engaged before tightening the foot bolt. Incorrectly engaged serrations can cause noise, looseness, and the eventual failure of the foot bolt.



7 Fitting Advice

934331 for snubber replacement

Symptoms	Cause	Solution
<ul style="list-style-type: none"> Sinking at heel strike Foot slap 	Heel too soft	<ol style="list-style-type: none"> Confirm A-P alignment and heel height accommodation Fit a harder ankle ball
<ul style="list-style-type: none"> Rapid transition from the heel strike through stance phase Difficulty controlling heel action (e.g. foot jars into mid-stance) Foot feels too rigid 	Heel too hard	<ol style="list-style-type: none"> Confirm A-P alignment and heel height accommodation Fit a softer ankle ball
<ul style="list-style-type: none"> Drop-off in mid-late stance Feels like walking downhill 	Forefoot too soft	<ol style="list-style-type: none"> Confirm A-P alignment and heel height accommodation Fit a harder snubber
<ul style="list-style-type: none"> Difficulty walking over the forefoot Feels like walking uphill 	Forefoot too hard	<ol style="list-style-type: none"> Confirm A-P alignment and heel height accommodation Fit a softer snubber


Note: Bilateral users may require stiffer snubbers than unilateral users. Trans-femoral users may require a softer plantarflexion action than trans-tibial users.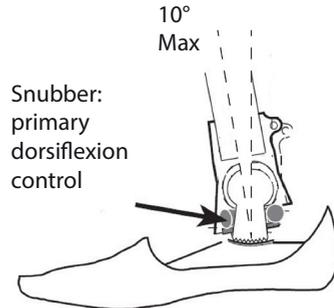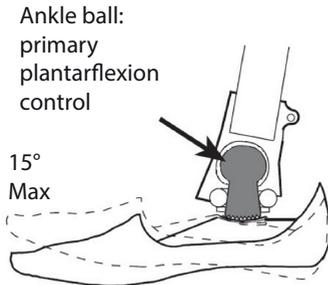

8 Technical Data

Operating and Storage Temperature Range:	-15 °C to 50 °C (5°F to 122 °F)
Component Weight:	201 g (7.1 oz)
Activity Level:	1–3
Maximum User Weight:	125 kg (275 lb)
Range of Adjustment:	360° rotation
Build Height:	60 mm
Proximal Attachment:	Blatchford Ø30 mm tube
Distal Attachment:	Blatchford foot

Fitting Length

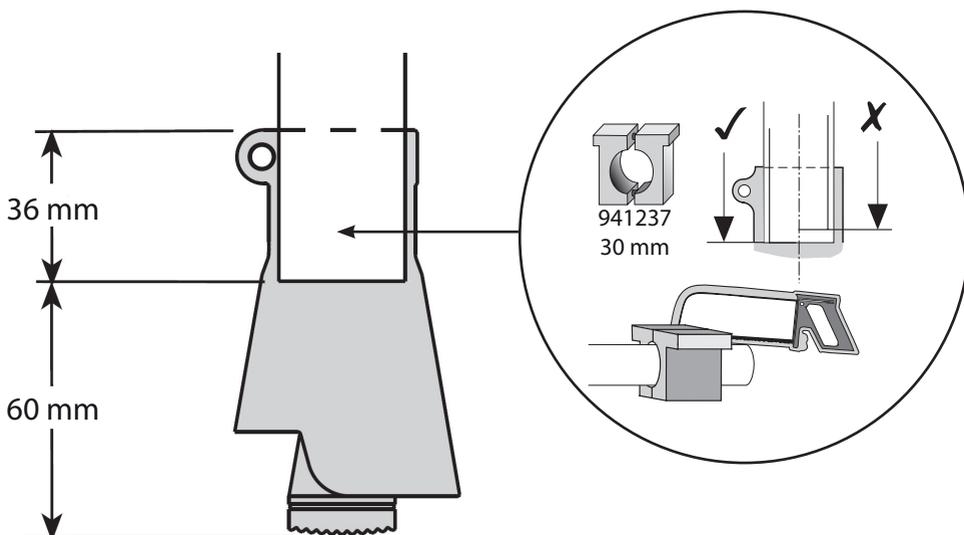9 Ordering Information

Part	Part number
Multiflex Ankle W	379342
Multiflex Ankle O	379343
Multiflex Ankle B	379344
Snubber 60° W	409007
Snubber 70° O	409008
Snubber 80 ° B	409009
Ankle ball W	409314
Ankle ball O	409315
Ankle ball B	409316
M6x20 Socket cap screw	910410
M6 Lock washer	916150
Collet	916150
Circlip	918000
Rubber and nylon lubricant	928107
Shin tube cutting jig 30 mm	941237

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class I Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty

This device is warranted for 24 months. The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions. See Blatchford website for the current full warranty statement.

Manufacturer's Registered Address

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Blatchford SAS
Endolite India Ltd