

Blatchford Products Ltd

Lister Road, Basingstoke, Hampshire,
RG22 4AH, United Kingdom
Telephone: +44 (0)1256 316600
Facsimile: +44 (0)1256 479705
Email: sales@blatchford.co.uk
www.endolite.co.uk

Pyramid Multiflex Ankle 125kg 409353 - 409355



endolite

get busy living

409353 409354 409355



60°
White



70°
Orange



80°
Blue

Standard Duty Snubber Kits

409007 409008 409009



60°
White



70°
Orange



80°
Blue



100 kg



125 kg

Structural Limitations



			Trans-tibial Only

60 kg

Max.

80 kg

Max.

100 kg

Max.



125 kg

Max.



This information is intended for prescription guidance only.
Structural limitations are shown above.

Notes:


Excessive dorsiflexion at ankle fit a harder snubber and vice versa.
Excessive plantarflexion at ankle fit a harder ball and vice versa.
Trans-femoral amputees may require a softer plantarflexion action than Trans-tibial.
Double amputees may require stiffer snubbers than single amputees.

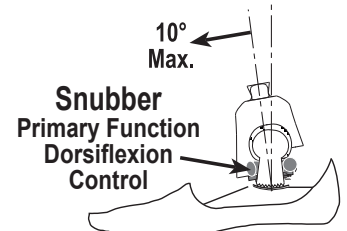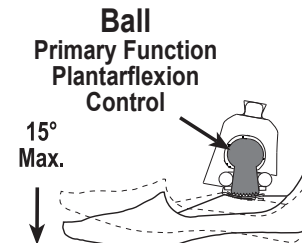


endolite

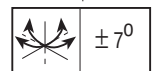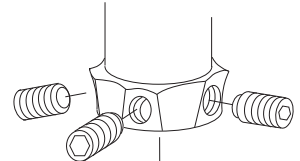


Mark label to show grade of ball assembly and snubber fitted. If grade of either is changed re-label (400014) accordingly.

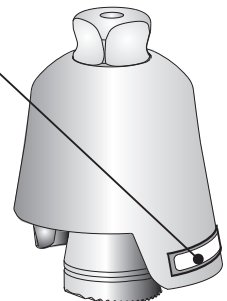

934331

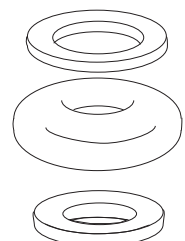

Recommended Alignment

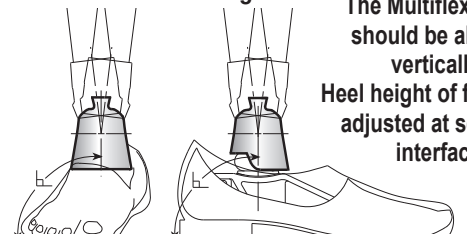

The Multiflex Ankle should be aligned vertically.
Heel height of foot to be adjusted at serrated interface

CE

938225/03-0411